A-Tam[®] Analtampons

aus PVA (Polyvinyl - Alkohol - Schaumstoff)

Hilfsmittel-Nr. 15.25.17.0001

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
A-Tam[®] anal tampons

made from PVA (polyvinyl alcohol foam)

INSTRUCTIONS FOR USE

Please also read the other notes/recommendations.


Please proceed as described below in words and pictures when inserting the tampon. The procedure is similar to inserting a suppository. Due to generally applicable hygiene regulations, this product is a single-use disposable product ☒. The applicator can be reused several times.

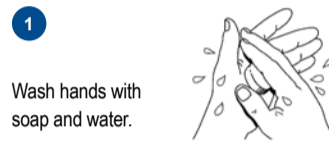
Wash hands with soap and water.


Clean the anal area thoroughly.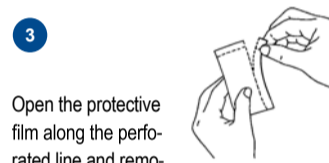


Open the protective film along the perforated line and remove the tampon.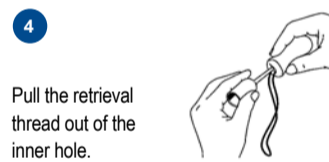


Pull the retrieval thread out of the inner hole.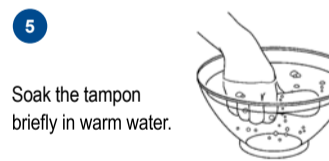

Soak the tampon briefly in warm water.

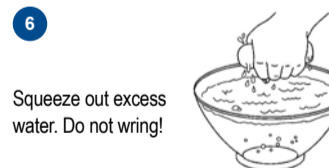Squeeze out excess water. Do not wring!

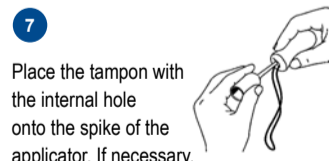Place the tampon with the internal hole onto the spike of the applicator. If necessary, Vaseline can be applied to the tampon tip to improve lubrication.

Use the applicator to insert the tampon far enough into the intestinal tract until only the retrieval thread is visible (see illustration).

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Notes/recommendations

- Empty bowels before insertion
- Use toilet training to get used to consistent bowel evacuation times
- Start with small sizes when you are using the product for the first time
- Wait for the pinch reflex (approx. 15 to 20 seconds) ***
- Remove the tampon after a maximum of 12 hours
- If the retrieval thread breaks during removal of the tampon despite frequent checks and you are unable to remove the tampon, please consult your doctor

***This reflex no longer occurs after an acclimation period of approx. 5-6 weeks

Faecal incontinence is caused by:

Spina bifida, Anal insufficiency, Rectal prolapse, Anal atresia, Multiple sclerosis, Paraplegia, Surgical damage, Deterioration of the sphincter muscles caused by radiotherapy, Age-induced deterioration of the sphincter muscles, Stoma

Contraindications/special instructions:

A-Tam® should not be used if there are known intolerances to PVA* foam.

A-Tam® is a disposable product and must not be reused, as this could lead to bacterial contamination.

Side effects

No known side effects

Material*: Polyvinyl alcohol foam - Thread: Cotton

Med SSE System GmbH does not accept any liability for damages arising from improper use.

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Med SSE System GmbH non si assume alcuna responsabilità per danni causati da un uso improprio.